



## **Instructions for use.**

### **Valid for FFP2 and MNS masks**

#### **Instructions for use protective masks**

Half masks to protect against solid and liquid particles.

Do not use against vaporizing aerosols and gases.

Choose products according to the professional scope.

Do not use in potentially explosive areas.

The masks are intended for single use.

Throw away the mask after use.

Immediately leave the area and remove respirators if breathing becomes difficult.

If dizziness or other discomfort occurs, as well as irritation, odor or other contamination notice, or if the respirators are damaged.

Do not modify or repair any part of the mask.

If damage occurs or the mask is soaked, remove the mask and replace it with a new one.

Do not use if the use-by date has expired (see packaging).

#### **Check before use**

Select the mask type according to the work requirement.

The manufacturer's instructions must be followed.

#### **Use and setting**

Take the half mask out of the packaging.

Make sure there are no holes in the breathing protection zone.

Fit the selected mask to your face as indicated on the original packaging Adjust the nose clip to the shape of your nose with both hands and exhale strongly to ensure that it is tight to consider.

Make sure the nose clip adapts to the shape of your nose and face.

When removing the mask, please DO NOT touch the mask, only the ear straps.

Facial hair and scars, if they are between the face and the mask, greatly increase the leakage, which leads to a reduction in protection

#### **Storage**

Store the mask according to the manufacturer's rules.

The room temperature of the storage area must be between -20 ° and 40 ° C. The humidity of the storage area must not exceed 80%.

#### **FERMAMED GmbH**

Ottostraße 1

84051 Essenbach

Germany

Info@fermamed.de